


UNITY® PLxtra
CENTRATION CHART

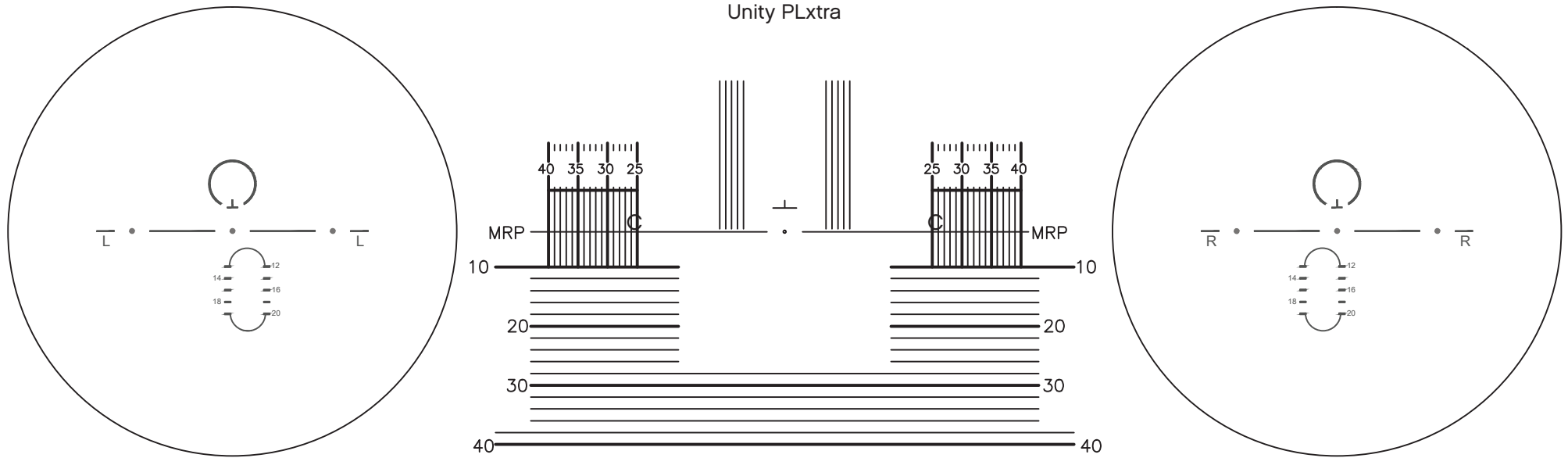


Engraving Index		
Product	Symbol	Fitting Designs
Unity PLxtra	PT	12, 14, 16, 18 & 20

Instructions for Fitting Guide use:

1. Measure patient's fitting height
2. Select fitting design based on patient needs

 Recommended designs

 Additional reading area needed

Unity PLXtra										
Fitting Designs	Unity PLXtra		Unity PLXtra		Unity PLXtra		Unity PLXtra		Unity PLXtra	
Fitting Height	Reading Area	Intermediate Area	Reading Area	Intermediate Area	Reading Area	Intermediate Area	Reading Area	Intermediate Area	Reading Area	Intermediate Area
12	5	7	—	—	—	—	—	—	—	—
13	6	7	—	—	—	—	—	—	—	—
14	7	7	5	9	—	—	—	—	—	—
15	8	7	6	9	—	—	—	—	—	—
16	8	7	7	9	5	11	—	—	—	—
17	8	7	8	9	6	11	—	—	—	—
18	8	7	8	9	7	11	5	13	—	—
19	8	—	8	9	8	11	6	13	—	—
20	8	—	8	9	8	11	7	13	5	15
21	—	—	8	—	8	11	8	13	6	15
22	—	—	—	—	8	11	8	13	7	15
23	—	—	—	—	—	—	8	13	8	15
24	—	—	—	—	—	—	8	13	8	15
25	—	—	—	—	—	—	—	—	8	15
26	—	—	—	—	—	—	—	—	8	15